

Getinge EDS8
- for safe drying and storage

GETINGE GROUP
Passion for life




Take control of your
flexible endoscopes

Safe – for patients, staff and your budget

Micro-organisms can grow to a dangerous level within a few hours, therefore unless your endoscope is dried quickly and protected against contamination a new treatment is required before it can be used on the next patient. The Getinge EDS8 drying and storage cabinet ensures that up to eight disinfected flexible endoscopes are always available for immediate use.

Ensures disinfected endoscopes remain microbiologically safe and ready to use

- HEPA filtered air continually circulates through the internal channels of the endoscope and over the exterior surface
- Proven independently to dry and maintain microbial integrity for up to 7 days
- Alarm function ensures adequate air flow and environmental conditions for safe storage of each individual endoscope
- Horizontal shelves prevent cross contamination between endoscopes
- Complies with EN 16442:2014
- Tested with three additional bacteria strains to comply with French Authorities (ANSM) recommendation for added safety

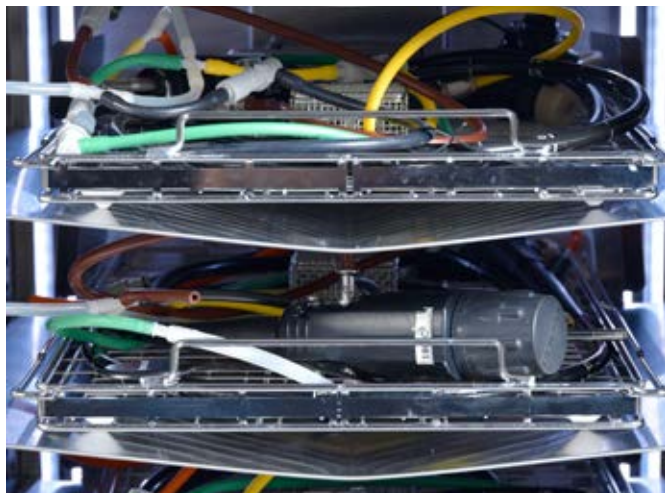

Horizontal shelves



Getinge EDS8 drying and storage cabinet



Foot pedal for hands-free operation

Protects against endoscope damage

- Common basket with Getinge AER's minimizes handling during transfer
- Hands-free door actuation allows operator to completely focus on the managing the endoscope that is being loaded/unloaded



User-friendly display

Efficient management of endoscope inventory

- User-friendly, graphical display provides name, status and location of all the endoscopes in the cabinet
- Secure door function limits cabinet access to authorized personnel (when enabled)
- Prepared for connection to T-DOC EndoTrace and T-DOC 2000 systems which allows for management of all endoscopes within the facility



RFID scanner for enhanced safety

Integral traceability of drying and storage function

- RFID scanner enables error-proof entry of endoscope ID and operator for recording purposes
- Built-in printer to provide paper copy of process record if desired
- T-DOC enabled link allows creation of complete endoscope reprocessing record that includes drying and storage phase as well as endoscope management at the procedure room with traceability to the patient

Because **all patients** deserve a clean
and disinfected scope **every time**

GETINGE GROUP

Getinge Infection Control AB
P O Box 69, SE-305 05 Getinge, Sweden
Phone: +46 10 335 00 00
info@getinge.com
www.getinge.com

MAQUET
GETINGE GROUP

ARJOHUNTLEIGH
GETINGE GROUP

GETINGE
GETINGE GROUP

Getinge Group is a leading global provider of innovative solutions for operating rooms, intensive-care units, hospital wards, sterilization departments, elderly care and for life science companies and institutions. With a genuine passion for life we build quality and safety into every system. Our unique value proposition mirrors the continuum of care, enhancing efficiency throughout the clinical pathway. Based on our first-hand experience and close partnerships, we are able to exceed expectations from customers – improving the every-day life for people, today and tomorrow.